EN

FILT'RAY Compact - STERILE DISPOSABLE WATER FILTERS

Duration of use: 1, 2 and 3 months

Shower Water Filter and Faucet Water Filter

ES

Filtros FILT'RAY Compact - FILTROS ESTÉRILES

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Filtros mango de ducha y grifo

PT

Filtros FILT'RAY Compact - FILTROS ESTÉREIS

Tempo de vida útil: 1, 2 e 3 meses

Filtros de chuveiro e de torneira

 : **AQUATOOLS** - 11, rue Lobineau - 75006 PARIS | France

Headquarters/Oficina central/Sede: **AQUATOOLS** - 18, rue du Maréchal Foch - 80130 Friville - France

Tel. : **+33 1 39 75 02 20**

US commercial subsidiary/Filial comercial EE. UU./Filial comercial EUA:
AQUATOOLS Inc - 1 Broadway, 14th floor, Cambridge MA 02142 United States

Tel: **+1 516 464 1513**

1 REFERENCES

FILT'RAY Compact Faucet Filter – AT20150US, AT20151US, AT20250US, AT20251US, AT20350US, AT20351US.

FILT'RAY Compact Shower Hand Water Filter – AT20161US, AT20261US, AT20361US.

To guarantee the optimal performance of the FILT'RAY Compact Disposable Water Filters, please ensure that you follow the instructions for installation (see 5) and read the warnings carefully (see 6).

2 GENERAL SPECIFICATIONS

- Disposable Sterile Water Filters.
- Effective against Bacteria including Opportunistic Waterborne Pathogenic Microorganisms.
- For up to 1, 2 & 3 months.

FILT'RAY Compact Filters are **Class Is medical devices in Europe, Class II in the USA.**

3 INDICATIONS FOR USE

The FILT'RAY Compact Disposable Water Filter is intended to be used to filter EPA (Environmental Protection Agency in USA) Quality Drinking Water. This microfiltration technology is effective for the reduction of **Healthcare-Associated Infections** by Opportunistic Waterborne Pathogenic Microorganisms. By retaining bacteria, the filters may aid in Infection Control. The filters produce water that is suitable for showering high risk-patients, washing neonates, drinking water and ice for immunosuppressed patients, superficial wound cleansing (minor cuts, scrapes, or abrasions), cleaning of equipment used in medical procedures and washing of surgeon's. **The filters are not intended to provide water that can be used as a substitute for USP grade sterile water.**

4 DESCRIPTION OF DISPOSABLE WATER FILTERS

The medical devices covered by this notice are filters for faucets and/or showers. They are commonly called "**FILT'RAY Compact tap filters**" for microfiltration cartridges and "**Screw-on shower filters**" for shower heads fitted with a microfiltration cartridge.

They are installed at the end of the drinking water network (faucet or shower) and serve as **physical barriers in the event of a bacterial alert** (*Legionella pneumophila*, *Pseudomonas aeruginosa*, and other Gram-negative bacilli, non-tuberculous Mycobacteria, fungi, protozoa and other particles from the water distribution network – absolute filtration grade of 0.2 µm) for an **indicative duration of 1, 2 or 3 months**. The protection of immunocompromised patients and vulnerable people is ensured thanks to the microfiltration cartridge. These medical devices are for single use only.

They are intended to be installed in hospitals or medical care facilities (retirement homes, specialized centers), as well as in collective facilities such as leisure or sports centers, hotels, etc.

Laboratory-based microbiological challenges of *Brevundimonas diminuta*, *Legionella pneumophila* and *Pseudomonas aeruginosa* on the FILT'RAY Compact demonstrate a **7-log retention capacity (see 9-2)** for all waterborne microorganisms larger than 0.1 µm nominal /0.2 µm absolute and qualify the filter as sterilizing grade² filter (**see 9-B**).

5 INSTALLATION INSTRUCTIONS

Warnings:

Wash and disinfect your hands before installing the filter to limit the risk of contamination or retro-contamination.

Check that the single packaging bag is not punctured or torn and that the filter is not damaged.

If necessary, dispose of the filter as its safety and performance are no longer guaranteed.

Check that the expiry date has not passed. If necessary, discard the filter because its safety and performance are no longer guaranteed.

Preliminary preparation of the taps:

- The screw-on shower filter: it is screwed quickly clockwise and until the seal is closed on all shower hoses complying with standard NF EN 1113.

The length of the hose must be adapted to avoid any contact between the shower filter and the ground. The gasket provided must be installed to ensure sealing and prevent possible leaks. Each time the shower head is changed, it should be replaced.

- The faucet filter: it must be fitted using the quick connectors recommended by AQUATOOLS. The quick connectors allow the filter to be connected to and disconnected from the faucet easily.

Visit the website or contact us for a list of available fittings.

General Information:

- The shower head filter (screw-on) features a standard fitting and can therefore be connected to all flexible shower hoses.
- The faucet and wall-mounted shower filters can be connected to most plumbing fixtures using an adaptor (clip-on); contact us for a list of commercially available adaptors.
- Recommended Storage: Between 5 and 35°C (41 and 95°F).

Installing:

1. Open the faucet to purge before installation to avoid early clogging.
2. Close or shut off the water supply to an electronic faucet.
3. When installing for the first time:
 - Faucet filter: place an adapter fitting.
 - Screw-on shower filter: remove the original shower head.
4. If replacing a worn filter: remove the filter.
5. **Wash and disinfect your hands.**
6. Partially open the unit packaging.
7. While keeping the filter in the bag to prevent contamination of the water outlet, screw the shower/hand spray filter to the end of the hose or clip the cartridge filter to the fitting adapter.
8. Remove the packaging.
9. Turn the water on for a few seconds to check that the filter and fittings remain perfectly sealed.
10. In case of leakage, repeat the procedure. Do not re-use a filter that has been connected.
11. Note the commissioning and the theoretical decommissioning date (commissioning date + theoretical duration of use) on the filter and on the traceability label.

Removal:

1. Close the water supply.
2. Release the used shower head filter by unscrewing it from the end of the hose or unclip the shower head filter or faucet filter from the adapter fitting.
3. Dispose of the filter as normal/household waste. Never re-use a filter that has been connected.
As the filter cannot be dismantled, the manufacturer cannot guarantee that the fibers are always in good condition.
Installation and use in a dirty or uncontrolled environment can be a source of retro-contamination.
Discard and replace any filter that appears visually soiled or contaminated.

However, use in a non-controlled environment can be a source of retro-contamination.

Any filter that is believed to be contaminated must be replaced, especially if the contaminant is visible.

To do so, filters must **not** be disconnected or soaked in a cleansing / disinfectant solution.

Waste management:

Used FILT'RAY Compact Disposable Water Filters do not present any special infectious risk. They are classified as normal household waste. The installation guide and the white carton can be recycled (paper/card) (see 9-1).

Maintenance:

To track maintenance procedures, a removable adhesive label (located on the filtration device) can be used to record important information for your safety records (date of installation, date of removal, filter unique identification number, etc.).

This information is also included in the UDI (Unique Device Identification) which is a datamatrix with information such as (01) GTIN: Global Trade Item Number, (10) Lot number, (17) expiration date, (21) Unit Number, (240) aqua-tools Part Number.

Manufacturing date, Expiration date and Lot number are also clearly identified on device labels and packaging.

The device label provides an additional space for the user to note clearly the date of first use and the date of disposal.

Duration of life is identified by a numerical symbol featuring the number of months (1, 2, 3) of the use-life specification.

6 WARNINGS AND CONTRAINDICATIONS

- Disposable Water Filters must not under any circumstances be used to produce water for **pharmaceutical purposes** (i.e., water that will be used in solutions intended to be administered orally, injected, or perfused).
- Disposable Water Filters **are not intended to provide water that can be used as a substitute for USP grade sterile water.**
- Disposable Water Filters are single-use (disposable). They cannot be sterilized or re-installed on another point of use.

PRECAUTIONS FOR USE

- Wash and disinfect your hands before touching or cleaning a filter.
- Purge the faucet before installing the filter to avoid early clogging.
- Do not use the filter if the packaging and/or the filter are damaged.
- Avoid knocks to or collisions with the filter. Replace if damaged.
- Discard and replace any filter that appears visually soiled or contaminated.
- Do not disconnect the filter while in use.
- The water outlet must not come into contact with skin or hair. In the case of contact, there is a risk of retro-contamination. It is advisable to replace the filter.
- If there is a decrease in water flow, it is advisable to replace the filter.
- Once disconnected, the filter should be discarded (single use).
- The microfiltration cartridge must not be adapted, modified or opened.
- The filter must not be used for the production of water for pharmaceutical use.
- The filter does not change the potability of the water, and it does not modify its concentration of salts, metals, scale, chemicals, pesticides or dyes.
- Shower filters must always be repositioned on the support after use.

7

TECHNICAL SPECIFICATIONS

Conditions of use:

The performance of FILT'RAY Compact filters is guaranteed when the pressure and temperature of the water in the network are respectively between 1 and 5 bar/41 °F and 140 °F. The measured outlet water flow rates are as follows:

FILT'RAY Compact filter - Typical Flow Rate (gal/min)					
	Water pressure (PSI)				
	14.5	29	43.5	58	72.5
Flow rate (gal/min) - Faucet 1M	0.78	1.14	1.43	1.69	1.92
Flow rate (gal/min) - Faucet 2M	1.20	1.72	1.98	2.24	2.44
Flow rate (gal/min) - Faucet 3M	1.20	1.69	1.98	2.21	2.44
Flow rate (gal/min) - Shower head 1M	1.56	2.29	2.91	3.33	3.38
Flow rate (gal/min) - Shower head 2M	1.61	2.39	2.91	3.22	3.43
Flow rate (gal/min) - Shower head 3M	1.69	2.5	3.07	3.43	3.67

FILT'RAY Compact filter - Typical Flow Rate (L/min)					
	Water pressure (bar)				
	1	2	3	4	5
Flow rate (L/min) - Faucet 1M	3	4.4	5.5	6.5	7.4
Flow rate (L/min) - Faucet 2M	4.6	6.6	7.6	8.6	9.4
Flow rate (L/min) - Faucet 3M	4.6	6.5	7.6	8.5	9.4
Flow rate (L/min) - Shower head 1M	6	8.8	11.2	12.8	13
Flow rate (L/min) - Shower head 2M	6.2	9.2	11.2	12.4	13.2
Flow rate (L/min) - Shower head 3M	6.5	9.6	11.8	13.2	14.1

Exposure to chemical and thermal treatments:

The filter can tolerate the following treatments:

- Heat treatment of 70°C (158 °F) for 30 minutes (maximum cumulative).
- Chemical treatment: active chlorine.
 - Filter 1M: 100 ppm for 2 hours.
 - Filter 2M: 100 ppm for 3 hours.
 - Filter 3M: 100 ppm for 4 hours.

Replace the filter if the chemical or thermal treatment exceeds these thresholds.

Clogging tolerance:

If the water in the drinking water network has an unusual rate of particles or microorganisms in suspension, it is possible to observe a decrease in the output flow rate before the theoretical decommissioning date (date of entry into service + theoretical duration of use).

When the flow rate becomes insufficient, replace the filter.

Warning: Clogging may occur before the theoretical discharge date, but it does not directly affect the patient.

Tip: In the case of early clogging, pre-filtration at different levels of the system is recommended to stop suspended solids and/or colloids.

8

PRODUCT COMPLIANCE

The Sterile FILT'RAY Compact Disposable Water Filters are sterilized by gamma irradiation in compliance with ISO 11137. Do not re-sterilize.

Sterile FILT'RAY Compact Disposable Water Filters:

- Are **Class II Medical Devices in the US** - FDA-510K Clearance.
- Are **Class Is Medical Devices for Europe** that comply with European Regulation EU 2017/745 CE Marking year 2023.
- Complies with Drinking Water Certifications (**ACS** - France, **KTW** - Germany, **WRAS** - United Kingdom).

9

MANUFACTURING INFORMATION

FILT'RAY Compact filters are manufactured by:

Registered Office: **AQUATOOLS** | 11, rue Lobineau | 75006 PARIS | France

Headquarters: **AQUATOOLS** | 18, rue du Maréchal Foch | 80130 FRIVILLE | France

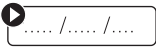
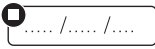
US address: **AQUATOOLS Inc** – 1 Broadway, 14th floor, Cambridge MA 02142 United States – Call us: +1 516 464 1513

1. For more information, visit our website: www.aquatools.us **E-mail:** sales@aquatools.us
2. The “Scientific and Technical Validation Guideline” relating to filters is available on request.
3. The paper version of the notices remains available on request.
4. Any serious incident must be notified to the manufacturer and to the competent authority of the Member State where the user and/or the patient resides.

Reference:

A sterilizing grade filter is defined in the "American Standard Test Method ASTM F838-20 and FDA - Guidance for Industry - Sterile Drug Products Produced by Aseptic Processing - Current Good Manufacturing Practice.

Product label - Meaning of symbols:

		
Catalog number	Batch Code	Expiration date
		
Device sterilized by irradiation	Do not resterilize	Do not use if the package is damaged
		
Manufacturer's address	Global Trade Item Number	Consult the instructions for use
		
Sterile barrier system only	Medical Device	Unique Device Identifier
		
Do not reuse	Manufacturing date	Shower head filter
		
Tap filter	Shower flow	Straight flow
		
Serial Number	Commissioning date	Decommissioning date
		
Use-by date after commissioning 1M: 1 month - 2M: 2 months - 3M: 3 months	Quantity	Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner
		
Name of the range		

 : AQUATOOLS - 11, rue Lobineau - 75006 PARIS | France

Headquarters/Oficina central/Sede: AQUATOOLS - 18, rue du Maréchal Foch - 80130 Friville - France
Tel. : +33 1 39 75 02 20

US commercial subsidiary/Filial comercial EE. UU./Filial comercial EUA:
AQUATOOLS Inc - 1 Broadway, 14th floor, Cambridge MA 02142 United States
Call us : +1 516 464 1513

FILT'RAY Compact Grifo - AT20150US, AT20151US, AT20250US, AT20251US, AT20350US, AT20351US.

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